










Shortcut Keys

of OnDemand3D Software

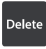


General

- Key Operation on Image Pane





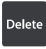


Key	Operation
	Overlay On / Off
	Cancel or Stop the current function
	Delete selected annotation/measurement
	Launch DBM
	Show or Hide all DSI(Dynamic Sensitive Item)
	Launch Window Preset Dialog in any module
	Capture selected pane without overlay action
	Minimize or Maximize selected pane
	Launch QLB dialog in any module

- Key Operation on Fine Tuning

Key	Operation
	Overlay On / Off





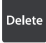







DBM Module

- Key Operation for Patient Study





Key	Operation
	Refresh Search List
  or 	Perform same as "+ button" in Patient list (Double click)
	Launch DLB
	Deselect from Patient list or clear value from Search box
	Delete patient list

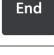3D & Dental Module





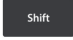


- Key Operation on MPR Image Pane

Key	Operation
	Move to Next Slice
	Move to Next Slice
	Move to Previous Slice
	Move to Previous Slice
	Move to Next Slice
	Move to Previous Slice (by 1/10 slices)
	Move to Next Slice (by 1/10 slices)
	Move to First Slice
	Move to Last Slice
	Implant Undo (Dental Tab)
	Implant Undo (Dental Tab)
	A/S/C Position Dlg open (Slice Direct Move) (Only on 3D Module)






• Mouse Left Button Operation on Image Pane

Key	MPR	3D / 3D-Zoom	Endoscopy
 + 	Zoom In	Zoom In	Zoom In
 + 	Pan	Pan	Pan

• Mouse Right Button Operation on Image Pane



Key	MPR	3D / 3D-Zoom	Endoscopy
	Windowing	Object Rotation	Rotation
 + 	Zoom Out	Zoom Out	Zoom Out
 + 	Zoom	Zoom	Zoom
 + 	Auto Windowing		

• Mouse Wheel Operation on Image Pane

Key	MPR	3D / 3D-Zoom	3D / 3D-Zoom	Endoscopy
	Move to Next/ Previous Slice	Rotate 3D Image (by 30 degrees)	Backward and forward movement	Scroll through
 + 	Move to Next/ Previous Slice (move by 1/10)	Rotate 3D Image (by 30 degrees)	Backward and forward movement (3 times faster)	Scroll through (3 times faster)
 + 	Move to Next/ Previous Slice (move by 1/10)	Rotate 3D Image (by 30 degrees)	Backward and forward movement (3 times faster)	Scroll through (3 times faster)

Report Module

- Key Operation for Reporting

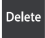Key	Operation
	Delete selected image from Thumbnail
	Refresh Database or History



Korea | **Cybermed Inc.**

Headquarters 6-26, Yuseong-daero 1205beon-gil, Yuseong-gu, Daejeon, Korea (34104)

Sales Office 1215, SJ Technoville 60-19 Gasan-Dong, Geumcheon-gu, Seoul, Korea (08511)

[**Contact**] **Tel.** +82.2.6959.1553 **Fax.** +82.2.6959.1554

E-mail. info@ondemand3d.com / support@ondemand3d.com

USA | **OnDemand3D Technology Inc.**

1382 Valencia Ave. Unit K, Tustin, CA 92780, USA

[**Contact**] **Tel.** +1.949.341.0623 **Fax.** +1.949.334.1317

E-mail. info@ondemand3d.com / support@ondemand3d.com